

**PGT BIOPSY LABELLING INSTRUCTIONS**

**PROVIDED COMPONENTS:**

- PGT Biopsy kit
- Unique labels with QR code
- Sample submission form

**INSTRUCTIONS:**


- The unique labels are in an alphanumeric format, with each clinic assigned a specific letter, followed by numbers. The letter assigned for your clinic will be emailed to you when we send the first batch of labels.
- Each unique tube label is provided in duplicates, as shown below:



- The top of the biopsy tube must be labelled in **PERMANENT PEN** with the patient initials and the embryo number.



- Thereafter, a unique label should be pasted horizontally onto the side of the PCR tube containing the embryo biopsy and the corresponding label on the sample submission form. **The label on the tube must always match the code of the label associated to that particular embryo on the sample submission form.**

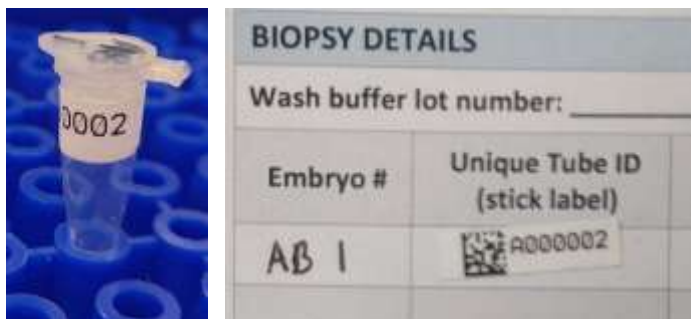

- In addition to the labels, the sample submission form should have the following information completed for each biopsy: Embryo number and initials, embryo grade, biopsy day (5/6/7), biopsy date, biopsy performed by, and loading performed by.
- A completed sample submission form should accompany the biopsies in the shipping box sent to Juno Genetics.